


UNPARALLELED TECHNOLOGY. POWERFUL RESULTS.


DUAL-ACTION MOA

The *Stealth 360* Peripheral Atherectomy System is uniquely designed to enable simultaneous modification of both intimal and medial calcium. The crown's diamond surface differentially sands intimal calcium^{1,2}, while pulsatile forces impact medial calcium.¹

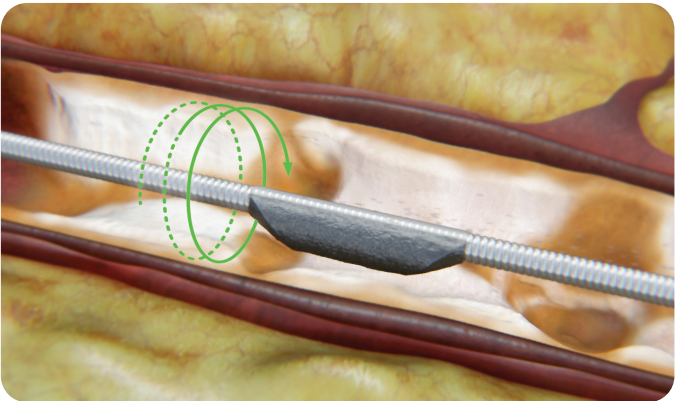

FEWER COMPLICATIONS³

Our studies demonstrated orbital atherectomy + low pressure PTA resulted in lower complications⁴ and less bailout stenting than PTA alone.³

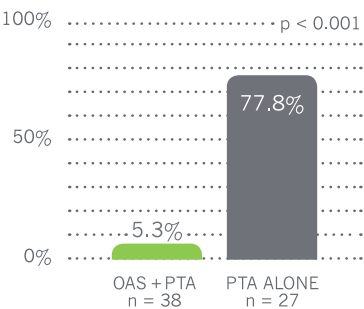

PERCENT OF LESIONS REQUIRING BAILOUT STENTS³

COMPLIANCE 360° Study: prospective, randomized, multi-center study comparing orbital atherectomy + PTA to PTA alone in calcified lesions above-the-knee (ATK).
Bailout stent: stent placed due to residual stenosis >30%.

CHOOSE THE RIGHT CROWN FOR THE LESION.

Devices are available in 3 unique crown designs in multiple sizes.

SOLID CROWNS

MODEL NUMBER	CROWN SIZE	SHAFT LENGTH	INTRODUCER SHEATH	QTY
PRD-SC30-125	1.25 mm	145 cm	6 Fr	1
PRD-SC30-150	1.50 mm	145 cm	6 Fr	1
PRD-SC30-200	2.00 mm	145 cm	6 Fr	1

CLASSIC CROWNS

MODEL NUMBER	CROWN SIZE	SHAFT LENGTH	INTRODUCER SHEATH	QTY
DBE-150	1.50 mm	145 cm	6 Fr	1
DBE-200	2.00 mm	145 cm	6 Fr	1

MICRO CROWN with GLIDEASSIST

MODEL NUMBER	CROWN SIZE	SHAFT LENGTH	INTRODUCER SHEATH	QTY
PRD-SC30-MICRO	1.25 mm	145 cm	4 Fr	1

Indications: The *Stealth 360* Peripheral Orbital Atherectomy System is a percutaneous orbital atherectomy system indicated for use as therapy in patients with occlusive atherosclerotic disease in peripheral arteries and stenotic material from artificial arteriovenous dialysis fistulae. Contraindications for the system include use in coronary arteries, bypass grafts, stents, or where thrombus or dissections are present. Although the incidence of adverse events is rare, potential events that can occur with atherectomy include: pain, hypotension, CVA/TIA, death, dissection, perforation, distal embolization, thrombus formation, hematuria, abrupt or acute vessel closure, or arterial spasm.

DISTRIBUTED BY:



For more information, please visit our website at www.OrbusNeich.com or contact us:

CORPORATE HEADQUARTERS Units 303 & 305, 3/F, Building 20E Phase 3, Hong Kong Science Park Shatin, N.T., Hong Kong T: +852.2802.2288 F: +852.2507.3532	EMEA HEADQUARTERS Drs. W. van Royenstraat 5 3871 AN Hoevelaken The Netherlands T: +31.33.254.1150 Toll Free Phone: 00800.0254.1150* *Only for Belgium, Denmark, Germany, Ireland, Netherlands, Norway, Sweden and UK
---	--

@2022 OrbusNeich Medical Company Limited or its affiliates. All rights reserved. G-70-2100 Rev01

MANUFACTURED BY:



CARDIOVASCULAR
SYSTEMS, INC.

1225 Old Hwy 8 NW St. Paul, MN 55112, USA

T: +1 651-259-1600 F: +1 612-677-3355
+1 877-274-0901 W: csi360.com

CSI, *Stealth 360*, *GlideAssist*, *ViperSlide* and *ViperWire Advance* are trademarks belonging to *Cardiovascular Systems, Inc.* ©2021 Cardiovascular Systems, Inc. EN-0023-OUS.A 0521

STEALTH 360

PERIPHERAL ORBITAL ATHERECTOMY SYSTEM



ADVANCING PAD TREATMENTS IN CALCIFIED LESIONS.

DISTRIBUTED BY:



MANUFACTURED BY:




¹ Adams GL, et al. J Cardiovasc Transl Res. 2011 Apr;4(2):220-9.
² Krishnan P, et al. J Endovasc Ther. 2017;24(1):167-68.
³ Dattilo R, et al. J Invasive Cardiol 2014;26(8):355-60.
⁴ Shammas NW, et al. J Endovasc Ther. 2012;19(4):480-88.

POWERFUL TECHNOLOGY FOR MORE PATIENTS


Having a greater impact on PAD starts with optimally treating patients. The Stealth 360 gives you the power to treat calcific plaque in peripheral arteries with our Dual-Action MOA uniquely designed to enable simultaneous modification of both intimal and medial calcium.¹



SOLID CROWN
For maximum calcium removal



CLASSIC CROWN
For vessel bends, ostial lesions and distal BTK procedures



MICRO CROWN WITH GLIDEASSIST*
For tortuous vessels and tight bends BTK

Convenient three-speed controls* allow quick adjustments within the sterile field

Easy, one-touch start button makes device power-up effortless

Sleek, electric-powered handle allows two-minute set-up and provides efficient torque transfer to the shaft and crown

Prime button allows quick flushing

GlideAssist for the 1.25 Micro Crown* (4 Fr) enables the crown to spin at a slow speed (5k RPM) for easier tracking and removal of the device

THE COMPLETE STEALTH 360 SYSTEM UTILIZES:

- OAS Pump
- ViperWire Advance
- ViperSlide
- Saline Line (comes packaged with the Stealth 360 device)

¹ CSI data on file.
*GlideAssist mode is only available on the 1.25 Micro Crown.